


Clear-IT™

Sample Marvelously with Elcam's new Closed Blood Sampling System

The Clear-IT™ Closed Blood Sampling System is an advanced tool designed by Elcam in response to two common problems associated with blood sampling: ICU Anemia due to blood waste associated with blood sampling, and infections due to exposure of patient's blood system to the atmosphere.

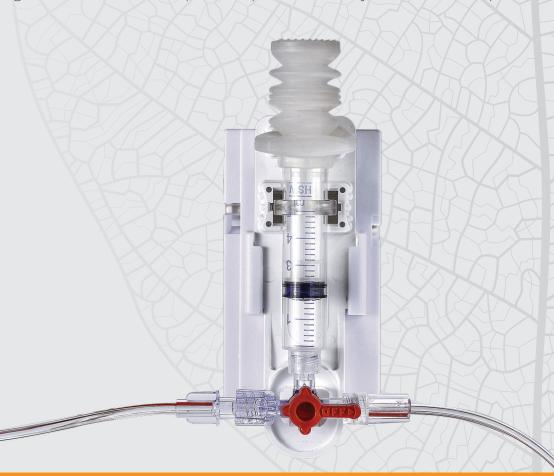

Patient Monitoring




Elcam Devices Elcam Stopcocks Elcam Accessories

Clear-IT™ Closed Blood Sampling System is designed for simplicity, safety and ease of use. The **Clear-IT™** combines an **In-Line** Reservoir for waste collection and a **stopcock with Marvelous™ handle** that enables flushing the line and syringe tip with minimal residual volume.

The **In-Line Reservoir** eliminates the discarding of waste taken through the set prior to the blood sampling. The entire clearing volume is reinfused to the patient upon completion of the sampling process in a safe manner that minimizes contamination. The Reservoir is designed to be connected to a standard mount plate.

The **stopcock with the Marvelous™ proprietary handle design** and its circumferential channel, assures clearing of the line and the syringe's tip with minimal unflushable (residual) volume after blood reinfusion following blood sampling.

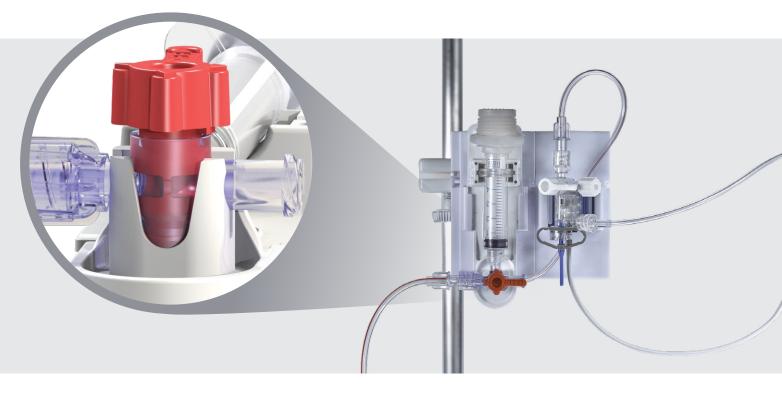

Benefits

- Intuitive & Easy to use Minimal training required
- Efficient Reduces blood waste by reinfusing clearing volume back into the patient
- Safe (*) Closed in-line design protects patient's blood from exposure to bacteria
 (*) Continuous line and syringe clearing decreases opportunities for bacterial contamination

The Clear-IT™ is available only as an OEM product

Elcam's Clear-IT™ Blood Sampling System is available as a stand-alone system. The system can be used in combination with Elcam's Sense- IT™ Integrated Disposable Pressure Transducer and with Elcam's Marvelous™ stopcock for minimal residual volume needleless blood sampling.



www.elcam-medical.com

Head Office / Manufacturer Elcam Medical

BarAm 1386000, Israel Tel: 972 4 698 8120/1 Fax: 972 4 698 0777 sales@elcam.co.il

USA Elcam Medical Inc.

1 University Plaza, Ste. 511 Hackensack, NJ 07601 Tel: +1 201 4571120 Fax: +1 201 4571125 info@elcam-medical.com

Europe Elcam Medical Europe

Via Emilia Romagna 15, 41012 Carpi (Modena) Italy Tel: +39 059 697710 Fax: +39 059 697705 sales.eu@elcam-medical.com

China Elcam China Office

Room 3309, Jinhai Mansion No. 89 of Anshun Road, Shanghai, PRC. Zip: 200051 Tel: +86 21 62829967 Fax: +86 21 62829967 johnliu02@126.com